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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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**Directions for Use**

Bleach n Smile Automix is a bleaching gel intended for medical in-office use only. The material is used by the dentist to brighten individual teeth. Bleach n Smile Automix is ideally suited to brightening vital teeth including endodontically treated teeth. The 4:1 automix syringe (cartridge) contains bleaching gel and activator. One dual syringe contains 5 g of material. The hydrogen-peroxide content of the ready-mixed gel is 35%.

**Designation of purpose**

A dental bleaching gel is a material used to brighten discolored teeth.

**Composition**

Water, hydrogen peroxide, potassium nitrate, sodium fluoride, sodium hydroxide, glycole derivative, thickener, dye.

**Indications**

- Brightening of discolored, devital teeth
- Brightening of discolored teeth caused by disease, by injury or iatrogenically, as an alternative to a crown or veneer
- Treatment of tooth discolorations caused by prescription drugs (e. g. tetracycline)

**Contraindications**

- Exposed dentin
- Defective filling margins
- Hypersensitive patients
- Allergy to or intolerance of the ingredients
- Pregnancy and lactation (due to insufficient data on the effects on pregnant and lactating women)

Some patients may experience slight sensitization in their teeth during or shortly after treatment, but this normally subsides after a short time.

**Stop treatment immediately if undue pain is being experienced.**

**Hazard statement**

The individual components (gel and activator) will irritate skin and mucous membranes, and they cause severe eye damage.

**Safety instructions**

Wear protective gloves/protective clothing/eye protection/face shield.
IN CASE OF EYE CONTACT: Gently flush with water for a few minutes. If possible, remove existing contact lenses. Keep on flushing. Immediately call a doctor.

**Interaction with other agents**

The dentist should consider known cross-reactions and interactions of the medical device with other materials already in the patient’s mouth before using the product.

**Preparations**

Determine the tooth color by comparing it with the Vita® Shade Guide. To evaluate the colors, please re-sort the Vita® Shade Guide as follows.

Values sorted by hue:
A1 A2 A3 A3,5 A4 B1 B2 B3 B4 C1 C2 C3 C4 D2 D3 D4

Sorted according to brightness value:

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B2-B1 = Values are increased twofold

A3-A2 = Values are increased fourfold

Use Air-Flow or a cleansing paste to clean the patient’s teeth. Use a cheek/lip retractor or similar instrument. Protect the gingiva (e. g. with Bleach ´n Smile Dental Dam or by applying a rubber dam). Do not anaesthetize the patient.

**Application**

First remove both the fixing clamp and the sealing cap, facing away from the patient and from yourself. Replace the sealing cap with a mixing tip. Now squeeze the gel from the syringe until a homogeneously colored mass with gel-like consistency is coming out of the tip. Discard the initial inhomogeneous mass. Storage for long periods at high temperatures may sometimes cause bubbles to form in the gel. This can also lead to inhomogeneous and more liquid mixtures, especially in the last quarter of the dual syringe. If this occurs, do not use the affected parts of the material. The gel should be applied in a thickness of about 1–2 mm to the labial surfaces of the teeth to be bleached. The point of the mixing tip is designed to spread the gel evenly.

To enhance the effect, you can use a standard plasma arc light or laser. **In this case, both the dentist and patient must wear safety goggles.** Please follow the manufacturer’s instructions. Do not overheat.

After a reaction time of about 10 minutes, aspirate the gel and wipe off the residue. If the bleaching effect is insufficient, the procedure can be repeated twice until the desired effect has been achieved. **Before each treatment, ensure that the gingiva protection is still sufficient.** Then determine the tooth color again.

After the final bleaching procedure, aspirate the gel, wipe it off and then rinse thoroughly with water. Now dry the tooth surface and polish with polishing paste. Subsequently, a remineralisation with After-Bleaching-Care or with a fluoride solution or fluoride gel is recommended. This is also suitable for treating pain if sensitization occurs.

If there is still enough material for another bleaching treatment, remove the mixing tip and replace the blue cap. Store the material in a refrigerator. Use a new mixing tip for the next treatment.

The patient should not eat, drink or smoke for 1 hour after treatment. Substances with strong colors (e. g. red wine, coffee) and foods containing acid should also be avoided in the days immediately following the treatment.

**Note:** Any composite fillings and restorative materials made of ceramic or metal are not bleached. Fillings should therefore be replaced 2 weeks after treatment.

**Devital or endodontically treated teeth**

The gel is also applied through an access opening on the lingual side of the tooth. First, check the height of the alveolar ridge. Then completely remove the filling material from the pulp chamber. Now apply one layer of glass ionomer filling with a thickness of 1–2 mm to the bottom of the pulp chamber on the same level with the alveolar ridge or coronally. First apply the gingiva protection to both the labial and the lingual sides of the tooth. Now, apply the bleaching gel to the labial surface as described under „Application“ above. Additionally, apply the gel through the access opening. Let the gel act for 10 min. Afterwards, carefully aspirate the gel and lightly rinse with water. If the bleaching effect is

insufficient, this procedure can also be repeated twice until the desired effect has been achieved. After the final bleaching procedure, aspirate the gel and rinse thoroughly with water. Carefully remove the gingiva protection. Next, close the access opening with a temporary filling material which does not contain eugenol. Wait two weeks before preparing the permanent filling.

**Shelf Life / Storage**

The maximum shelf-life is printed on the label of each syringe.
Store at 3 °C - 9 °C (37 °F - 48 °F). Do not freeze.
Do not use Bleach´n Smile after the expiration date, as a sufficient bleaching effect can then not be guaranteed any longer.

**Side-effects**

Adverse side effects of this medical device are extremely rare when properly processed and used. Any serious incidents arising in connection with the use of this product must be reported to the manufacturer indicated below and to the relevant competent authority.

**Information on disposal**

Residual quantities and packaging material must be disposed of in accordance with local and/or statutory regulations.

**Possible sources of error and their causes**

Insufficient bleaching effect	Repeat the bleaching procedure (several times if necessary)
Patient is experiencing pain	The gel has come into contact with exposed dentine or a loose filling (please cover before treatment). The patient is particularly sensitive (depending on the individual tooth structure).
There are white spots on the enamel after the bleaching procedure	Local demineralization or dehydration. Reversible. Disappears after fluoridation.
Gingiva is colored white after bleaching (process is reversible)	The gingiva is not adequately protected, or the protection is untight
The gel from the cartridge has an in-homogeneous color (colorless/red)	The gel has air bubbles in it as a result of being too stored at too high a temperature or for too long
Reduced composite-enamel adhesion after bleaching procedure	Peroxide residue. Leave for 1 -2 weeks before restoration.

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