# Orthosin Uni

---

CE 0297

**SCHÜTZ**  **DENTAL**  
Micerium Group

Schütz Dental GmbH · Dieselstr. 5-6 · 61191 Rosbach/Germany  
Tel.: +49 (0) 6003 814-0 · Fax: +49 (0) 6003 814-906  
info@schuetz-dental.de · www.schuetz-dental.de

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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## Directions for use

### General product description:

- Cold curing acrylic on methyl methacrylate basis for the production of orthodontic appliances.
- Suitable for the sprinkling and kneading techniques.

### Composition:

- Powder: pearl polymer made of polymethyl methacrylate (PMMA), pigments, initiators, benzoyl peroxide
- Liquid: methyl methacrylate (MMA), 1,4 butanediol dimethacrylate, initiators, stabilizers

### Indications:

- For the production of removable or fixed orthodontic appliances, bite splints and reflex anti-grinding splints in the sprinkling or kneading techniques
- Repairs and relinings of existing appliances or splints

### Contraindications:

Do not use in case of known allergies against one of the components.

### Application / processing:

#### Preparatory work:

Produce a plaster die and soak in water sufficiently before producing the appliance. Insulate the surface of the plaster die with an alginate insulation agent (e. g. FuturaSep insulation agent by Schütz Dental) according to the manufacturer's instructions.

#### Sprinkling technique:

Sprinkle the powder onto the die with the powder dispensing bottle, then wet the powder with Orthosin Uni liquid. Once all liquid has been soaked up and a homogenous dough has formed, the next layer can be applied. The dough is shaped layer by layer, as desired.

We recommend to wet all surfaces of the finished appliance once again with Orthosin

Uni liquid. Once the top layer loses its shine and appears matt (a thin skin will form during this process), the appliance must be entered in the pressure polymerization unit.

#### Polymerization in a pressure polymerization unit:

The polymerization time in a pressure polymerization unit is 15 min. at a pressure of 2 - 5 bar and at a water temperature of 45 °C.

#### Kneading technique:

Mix powder and liquid in a mixing cup with a ratio of 2.5 ml : 1 ml. After a soaking time of 4 - 5 min., apply the mixture directly to the die.

#### Polymerization in a pressure polymerization unit:

The polymerization time in a pressure polymerization unit is 15 min. at a pressure of 2 - 5 bar and at a water temperature of 45 °C.

#### Repairs and relining of existing appliances:

Produce a die or a stent, depending on the construction.

Clean the acrylic surfaces to be repaired, roughen them and smooth out the edges. Wet with Orthosin Uni Liquid or with HS Cross-Liquid (Schütz Dental). Mix acrylic according to the mixing ratio and let the acrylic flow in.

#### Finishing the material:

After polymerization, lift the appliance from the die and finish with suitable tools.

Silicone polishers (wheel, disc, roll) as well as hard metal mills or diamond grinding tools are suitable for finishing the material.

Prepolish with sandpaper or pumice.

#### Polishing:

Polish the material with goat hair bristle brushes, polishing paste and woolen wheels. A careful surface treatment and polish is a prerequisite for an optimal result.

## Troubleshooting / FAQ list:

<b>Error</b>	<b>Cause</b>	<b>Remedy</b>
acrylic does not cure properly	incorrect mixing ratio	<ul style="list-style-type: none"><li>– observe instructions for use</li><li>– keep mixing ratio</li></ul>
acrylic shows a whitish tint along the bottom	<ul style="list-style-type: none"><li>– insufficient insulation</li><li>– die was not soaked in water</li></ul>	<ul style="list-style-type: none"><li>– insulate sufficiently</li><li>– check insulation agent, if necessary</li><li>– soak die in lukewarm water for ca. 10 min.</li></ul>
acrylic forms bubbles along the bottom	<ul style="list-style-type: none"><li>– die was not soaked in water</li></ul>	soak die in lukewarm water for ca. 10 min.
acrylic surface appears white / porous	injection of the acrylic too late or acrylic put into pressure unit too late	observe processing times
excessive shrinkage	<ul style="list-style-type: none"><li>– too much monomer</li><li>– soaking time too short</li></ul>	<ul style="list-style-type: none"><li>– observe instructions for use</li><li>– use less monomer</li></ul>
acrylic fractures when lifted	improperly insulated	check insulation
splint fracture	design too thin / unsuitable	ensure a sufficient layer thickness (minimum 2 - 3 mm)

**Guarantee:**

Our processing instructions, regardless of whether they are issued verbally, in writing or in the form of a practical demonstration, are based upon our own experience and can therefore be regarded solely as guidelines. Our products are subject to continuous development. We therefore reserve the right to make possible changes.

**General notes:**

For deviations from 21 °C room temperature: Higher temperatures reduce the above times, lower temperatures increase them.

To further reduce the risk of intolerance reactions, we recommend storing the orthodontic appliance in lukewarm water for at least 12 hours before insertion.

**Storage:**

Storage temperature at 10 - 25 °C / 50 - 77 °F.  
Close container carefully.

**Shelf-life:**

The maximum shelf-life is printed on the label of each packaging. Do not use after the expiration date.

**Side-effects:**

With proper use of this medical device, unwanted side-effects are extremely rare. Reac-

tions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

Contains dibenzoyl peroxide.

**Contra-indications / interactions:**

If a patient has known hyper-sensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. In such cases, the composition of the medical device we supply is available upon request. The dentist should consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

**Note:**

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

**Disposal:**

Powder can be disposed of as domestic waste. Liquid must be disposed of as hazardous waste.

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