PrimeBond7

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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Instructions for use

1. Description
PrimeBond7 is a single-component, self-etching, light-cured dental adhesive which enables filling composites to be safely and reliably bonded onto natural dental enamel. The adhesive can be used in both self-etch and selective-etch techniques.
– PrimeBond7 is compatible with all conventional light-cured composite materials.

2. Indications
– Direct, light-cured composite restorations onto natural teeth.
– Indirect restorations made from composite and ceramic (inlays, onlays, veneers, crowns) with light-cured and dual-cured luting composites e.g. with Alphalink Cem.
– Repairs to fractured composite fillings.

3. Contraindications
– Do not bring into contact with exposed pulp tissue.
– Do not use if there is insufficient lighting (e.g. when fastening root canal posts).

4. Composition
Phosphate ester, carboxylic acid ester, methacrylate, photoinitiators, water, ethanol, fumed silica.

5. Use of PrimeBond7 in Direct Restorations 5.1. Preparation
Prepare the cavity by applying the usual techniques used in adhesive dentistry.
It is important to prevent saliva or blood from contaminating the cavity to ensure optimal results.
It is strongly recommended that the area is kept as dry as possible by using a rubber dam. Apply an underfilling material if necessary (e.g. glass ionomer cement).

Protecting the Pulp
In the case of very deep cavities, the area around the pulp must be coated with a thin layer of calcium hydroxide underfilling material and coated with a suitable material (e.g. glass ionomer cement). Leave the rest of the cavity open to „bond“.

5.2. Pre-Treating the Tooth Selective Enamel Etching
Apply the etching gel (37% phosphoric acid gel) carefully onto the edges of the enamel and leave for 30 seconds. Rinse the gel thoroughly and dry with oil-free compressed air. Enamel bonding can be further improved through selective enamel etching. Unprepared enamel must be conditioned with phosphoric acid. Please observe the instructions for use on the phosphoric acid etching gel.

Self-Etch Process
It is not necessary to etch once again with etching gel.

5.3. Applying the Adhesive
The surface that is going to be conditioned must not be touched or contaminated with saliva before PrimeBond7 is applied. Place a sufficient amount of PrimeBond7 into a mixing tray and apply with a disposable applicator. Continue using the material quickly (**within a max. of 3 minutes**). The bottle must be carefully resealed immediately after use. Apply the material over the entire cavity wall and leave for at least 20 seconds. This is the minimum amount of time that the material should be left. Remove any excess material with oil-free compressed dry air until it is not possible to see any freely moving liquid film. If the area becomes contaminated, it is necessary to clean, condition and use compressed dry air all over again. Next, light cure the adhesive following the instructions provided in the table.

Typ	Light Intensity	Polymerisations Time
Halogenlampe <p>LED lamp</p>	500 mW/cm² - 1200 mW/cm²	10 seconds

5.4. Applying the Composite Material
Apply the composite material following the applicable manufacturer’s instructions for use.

6. Applying PrimeBond7 for Indirect Restorations from Composite or Ceramic (Inlays, onlays, veneers, crowns)

The luting composite must be completely light cured after inserting the indirect restoration. 6.1. Preparing the Cavity
Prepare the cavity by applying the usual techniques used in adhesive dentistry. In the case of very deep cavities, the area around the pulp must be coated with a thin layer of calcium hydroxide underfilling material and coated with a suitable material (e.g. glass ionomer cement). Leave the rest of the cavity open to „bond“.

6.2. Checking the Fit and the Occlusion of the Restoration
In the case of delicate restorations, it is necessary to check the fit very carefully before bonding in order to prevent fractures. If necessary, it is possible to make corrections using a fine diamond tool operated at medium speed and only applying gentle pressure. Clean using a cleaning solution such as ethanol and dry. It is important to avoid cleaning the surfaces with phosphoric acid as this can weaken the adhesive bond. Coat the surfaces with a thin layer of PrimeBond7. Next, follow the process described under [point 6.6](#) (Applying the Adhesive).

6.3. Composite Surface Treatment
Dry and prepare the inner surfaces of the restoration by sandblasting and using cleaning solutions such as ethanol. It is important to avoid cleaning the surfaces with phosphoric acid as this can weaken the adhesive bond. Coat the surfaces with a thin layer of PrimeBond7.

6.4. Ceramic Surface Treatment
Prepare in accordance with the manufacturer’s instructions for use.

6.5. Pre-Treating the Tooth Selective Enamel Etching
Apply the etching gel carefully onto the edges of the enamel and leave for 30 seconds. Rinse the gel thoroughly and dry with oil-free compressed air. Enamel bonding can be further improved through selective enamel etching. Unprepared enamel must be conditioned with phosphoric acid. Please observe the instructions for use on the phosphoric acid etching gel.

Self-Etch Process
It is not necessary to etch once again with etching gel.

6.6. Applying the Adhesive
The surface that is going to be conditioned must not be touched or contaminated with saliva before PrimeBond7 is applied. Place a sufficient amount of PrimeBond7 into a mixing tray and apply with a disposable applicator. Continue using the material quickly (**within a max. of 3 minutes**). The bottle must be carefully resealed immediately after use. Apply the material over the entire cavity wall and leave for at least 20 seconds. This is the minimum amount of time that the material should be left. Remove any excess material with oil-free compressed dry air until it is not possible to see any freely moving liquid film. If the area becomes contaminated, it is necessary to clean, condition and use compressed dry air all over again. Next, light cure the adhesive following the instructions provided in the table.

9. Storage Instructions
Close the bottle tightly immediately after use. Do not use beyond the stated expiry date (see the label or packaging):
Storage temperature: 2 - 25 °C
If the product is not used for an extended period, it is recommended that it is stored in a refrigerator. Avoid contaminating the contents of the bottle by avoiding the use of contaminated disposable brushes.

10. Safety Warning
May cause allergic skin reaction. Harmful to aquatic organisms, with long lasting effects.

11. Safety Instructions
Avoid inhaling vapour / aerosol. Wear protective gloves / eye protection / face protection.
IN CASE OF CONTACT WITH THE SKIN: wash thoroughly with plenty of water. If the product causes skin irritation or a rash, seek medical advice / attention.

12. Side Effects
Adverse effects from using this medical product correctly are extremely rare. However, immune responses (e.g. allergies) or discomfort in specific areas cannot be completely ruled out in theoretical terms. Please inform us if you experience any unwanted side effects, even in cases of doubt.

13. Contraindications / Interactions
If the patient is hypersensitive to any of the ingredients, this product should not be used or may only be used under the strict supervision of the attending physician / dentist. In these cases, we can provide detailed information on request regarding the composition of the medical product that has been supplied. The dentist must take into account any known cross-reactions or interactions between this medical product and other materials in the mouth when this product is used. Phenolic substances (e.g. eugenol, wintergreen oil) inhibit the product’s ability to polymerise. We therefore recommend that underfilling materials that are used do not include these types of substances. Alkaline abrasives can have an adverse effect on Prime Bond7.

Typ	Light Intensity	Polymerisations Time
Halogenlampe <p>LED lamp</p>	500 mW/cm²- 1200 mW/cm²	20 seconds per surface

6.8. Finishing Off the Completed Restoration
Remove the retraction thread. Check the occlusion and functional movement and correct if necessary. Polish the edges of the restoration.

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