Temdent Classic

SCHÜTZ  **DENTAL**
Micerium Group

Schütz Dental GmbH • Dieselstr. 5-6 • 61191 Rosbach/Germany
Tel.: +49 (0) 6003 814-0 • Fax: +49 (0) 6003 814-906
www.schuetz-dental.de • info@schuetz-dental.de

CE 0297

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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Instructions for use

General product description

- Temdent Classic is a powder and liquid system for the production of temporary crowns and bridges.
- Temdent Classic is an autopolymer on polymethyl methacrylate basis.
- Temdent Classic is suitable for both the direct and the indirect method.

Composition

- Powder: bead polymer made of polymethyl methacrylate (PMMA), pigments, initiators.
- Liquid: methyl methacrylate, tetramethylene dimethacrylate, diurethane dimethacrylate, initiators, stabilizers.

Indications

- For the production of temporary crowns and bridges at a dental lab (indirect method)
- For the production of temporary crowns and bridges at a dental clinic (direct method)
- Repairs and relinings of existing restorations

Contraindications / interactions with other agents

Do not use in case of a known allergy against one of the components.

Mixing and adjusting the color

Temdent is mixed in a cup.

The consistency must be creamy liquid or doughy. This corresponds to a ratio of approximately 1.8 - 2.2 g powder and 1 g liquid (mixing ratio ca. 2:1).

First, fill the liquid into the cup. For an average crown, ca. 8 drops (0.15 g) and 0.3 g powder are necessary. By mixing the light and dark powders, the color is adjusted individually.

Application / Processing indirect method Preparatory tasks

Create a model from hard plaster (class IV). Set up the teeth in wax for an aesthetic and morphological analysis.

Next, create a silicone stent.

Instead of using the stent technique, it is also possible to pull a matrix foil over the model with help of a thermoforming device.

Insulate the plaster surfaces with an alginate-based insulation agent (Futura Sep, Schütz Dental), according to the manufacturer's instructions.

Insulate the situ model with silicone spray.

Cut off excess material at the margins. We recommend the use of a conventional thermoforming foil to stabilize thinly tapering margins.

Mixing ratio

1 weight part powder, 1 weight part liquid.

The consistency should be creamy without bubbles.

Production of the temporary restoration

Mix Temdent Classic (according to the mixing ratio) and fill it into the stent or into the matrix. The time from addition of the powder to the liquid to the application must not exceed 2 minutes. The matrix foil must be accurately repositioned on the model to avoid bite elevation and thus subsequent corrections in the mouth.

After ca. 6 min., carefully lift the temporary restoration from the model with the matrix. The elastic phase of ca. 1 min. and 30 sec. starts now. During this time, excess material on the margins can be removed with an instrument. Leave the temporary restoration on the model for final curing (ca. 2 min.). After final curing, grind and polish the margins.

Temdent Classic can be repaired.

If desired, the dentin core can be finished after the dentin casting. In order to improve the aesthetic effects, the surfaces are layered anatomically.

Therefore, the dentin core is ablated according to the tooth anatomy and then contoured. The result is checked with the matrix.

Next, completely coat the dentin core with enamel material. Use the same casting technique with the stent.

Finishing the material

After polymerization, lift the restoration from the model and finish with suitable tools.

Silicone polishers (wheel, lentil, roll) as well as hard metal cutters or diamond grinding tools are suited for finishing the material.

Prepolish with sandpaper or pumice.

Use / Processing direct method Impression technique

Before preparing the tooth, take an alginate or silicone impression and store it as usual.

After preparation, mix Temdent Classic as described above.

Let the mixed Temdent Classic flow into the desired area of the impression. The working range is 2 min. Ensure that the prepared stumps are properly insulated (e. g. with Vaseline), before applying the impression filled with Temdent Classic inside the patient's mouth.

Then apply Temdent Classic with the impression inside the mouth.

Temdent Classic cures slowly. After ca. 3 min. and 30 sec., the temporary restoration can be removed in its rubber-elastic phase. This measure prevents a sudden bonding of Temdent Classic with the prepared teeth. If the restoration sticks to the preparation, carefully remove it and set it back into the impression for curing. After curing, the restoration is dimensionally stable and can be finished.

Matrix technique

As described above, thermoform a 0.6 - 0.8 mm thick foil over a situ model insulated with silicone spray.

Mix Temdent Classic as described above and pour it into the matrix. The time from addition of the powder to the liquid to the application must not

exceed 2 minutes. The matrix foil must be repositioned exactly on the tooth row. The patient closes the tooth row, no need for subsequent grinding in the mouth. After 3 min. and 30 sec., carefully lift the temporary restoration from the mouth with the matrix. Further processing is as described above.

Polishing

Polish the material with goat hair brushes, polishing paste and soft woolen wheels. A careful surface treatment and polish are a precondition for an optimal result.

Repairs and relinings of existing restorations

Create a matrix or a stent, depending on the construction.

Clean the acrylic surfaces to be repaired, roughen them, smooth the edges and coat the surfaces with monomer or HS Cross-Liquid (Schütz Dental). Mix acrylic according to the mixing ratio and let it flow in.

Finish as described above.

Troubleshooting / FAQ list

Problem	Cause	Remedy
acrylic does not cure properly	incorrect mixing ratio	<ul style="list-style-type: none">– observe instructions for use– observe mixing ratio
acrylic shows a whitish tint along the bottom	<ul style="list-style-type: none">– insufficient insulation– model was not soaked in water	<ul style="list-style-type: none">– insulate properly– check insulation agent, if necessary
acrylic cures too fast/ too slow	unfavorable ambient temperature high temperatures speed up polymerization, low temperatures slow polymerization	adapt working method
incorrect color	powder light and powder dark can be mixed individually	chose a different mixing ratio
temporary restoration sticks to the preparation	residence time in the mouth too long	observe instructions for use
acrylic forms bubbles along the bottom	<ul style="list-style-type: none">– model was not soaked in water	soak model in lukewarm water for ca. 10 min.
color deviations of repairs	incorrect mixing ratio	observe mixing ratio
acrylic fractures when lifted	improperly insulated	check insulation
acrylic / prosthesis shows whitish streaks	<ul style="list-style-type: none">– improperly mixed– powder content too high	<ul style="list-style-type: none">– mix acrylic dough thoroughly with a spatula– observe mixing ratio

General notes

To further reduce the risk of intolerance reactions, we recommend storing the restoration in lukewarm water for at least 12 hours before insertion.

Storage

Storage temperature at 10 - 25 °C / 50 - 77 °F.
Close container carefully.

Shelf-life

The maximum shelf-life is printed on the label of each packaging. Do not use after the expiration date.

Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (e. g. allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

Contra-indications / interactions

If a patient has known hyper-sensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

Note

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

Disposal

Powder and liquid must be disposed of as hazardous waste.

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