# Tizian Blank Wax

---

Schütz Dental GmbH · Dieselstr. 5-6 · 61191 Rosbach/Germany  
Telefon +49 (0) 6003 814-0 · Telefax +49 (0) 6003 814-9 06  
[info@schuetz-dental.de](mailto:info@schuetz-dental.de) · [www.schuetz-dental.de](http://www.schuetz-dental.de)

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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## Handling instructions

### Description / Indications

Tizian Blank Wax are synthetic wax blanks, the purpose of which is to facilitate the manufacture of residue-free combustible wax-ups. These models can be used as dead-mould castings in the area of pressed-ceramics and in moulding technology, for example in the manufacture of crowns and bridges. In addition, wax-ups can be made simply from „Tizian Blank Wax“. The blanks are available in various geometric shapes and colours, for example grey and white.

### Important note

Tizian Blank Wax should not be used in the mouth.

### Physical properties

Chemical composition:	synthetic wax
Shore D hardness:	53–58
Density:	0.91–0.93 g/cm <sup>3</sup>
Flash point:	> 200 °C
Softening point:	101–109 °C

### Handling:

As usual, the designs are modelled using Tizian CAD/CAM and reamed out from Tizian Blank Wax.

The reamed wax frames can be processed using cross-toothed carbide milling cutters or cutting discs suitable for use with wax.

### Sprueing

The dimensions and positioning of the casting sprues depends on the alloy and/or the press ceramic. Please refer to the information provided by the relevant manufacturers.

### Investing / Casting

Please refer to the information provided by the manufacturer of the investment material and/or alloy.

### Storage

The material should be stored in a dry place where the temperature is between 10 and 25 °C. Protect from direct sunlight and moisture.

### Disposal

Tizian Blank Wax consist of synthetic wax. Leftover material can be used for other purposes where wax may be required. The material can be disposed of with the rest of the household waste.

### Guarantee

The application-specific information which we provide, whether imparted orally, in writing or through practical tutorials, is based on our own experiences and should therefore be seen only as a guide.

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